

## Latex Turbidimetry

### PRODUCT CODE TL001

#### INTENDED USE

The ASO-Turbilatex is a quantitative turbidimetric test for the measurement of ASO in human serum or plasma.

#### CLINICAL SIGNIFICANCE

Streptolysin O (SLO) is a toxic immunogenic exoenzyme produced by  $\beta$ -hemolytic Streptococci of groups A, C and G. Measuring the ASO antibodies are useful for the diagnostic of rheumatoid fever, acute glomerulonephritis and streptococcal infections. Rheumatic fever is an inflammatory disease affecting connective tissue from several parts of human body as skin, heart, joints etc.... and acute glomerulonephritis is a renal infection that affects mainly to renal glomerulus.

#### PRINCIPLE

Latex particles coated with streptolysin O (SLO) are agglutinated when mixed with samples containing ASO. The agglutination causes an absorbance change, dependent upon the ASO contents of the patient sample that can be quantified by comparison from a calibrator of known ASO concentration

#### REAGENTS

<b>Diluent (R1)</b>	Tris buffer 20 mmol/L, pH 8, Preservative.
<b>Latex (R2)</b>	Latex particles coated with Streptolysin O, pH 10 Preservative.
<b>ASO-CAL</b>	Calibrator. Human serum. ASO concentration is stated on the vial label.

#### PREPARATION

**ASO Calibrator:** Reconstitute with 1.0 mL of distilled water. Mix gently and incubate at room temperature for 10 minutes before use.

#### STORAGE AND STABILITY

All the kit components are ready to use, and will remain stable until the expiration date printed on the label, when stored tightly closed at 2-8°C and contaminations are prevented during their use.

**Do not freeze: frozen reagents could change the functionality of the test.**

Always keep vials in vertical position. If the position is changed, gently mix to dissolve aggregates that may be present.

**Reagent's deterioration:** Presence of particles and turbidity.

#### PRECAUTIONS

Components from human origin have been tested and found to be negative for the presence of HBsAg, HCV, and antibody to HIV (1/2). However, handle cautiously as potentially infectious.

#### SPECIMEN AND SAMPLE PREPARATION

Fresh serum or plasma, Stable 7 days at 2-8°C or 3 months at -20°C. Samples with presence of fibrin should be centrifuged.

Do not use highly hemolyzed or lipemic samples.

#### PROCEDURES

- Bring the reagents and the photometer (cuvette holder) to 37°C.
- Assay conditions:  
Wavelength: 540 nm (530-550)  
Temperature: 37 °C  
Cuvette light path: 1 CM
- Adjust the instrument to zero with distilled water.
- Pipette into a cuvette:

Diluent R1	800 $\mu$ L
Latex R2	200 $\mu$ L
Calibrator or sample	10 $\mu$ L

- Mix and read the absorbance immediately ( $A_1$ ) and after 2 minutes ( $A_2$ ) of the sample addition

#### CALCULATIONS

$(A_2 - A_1)$  sample  
 $(A_2 - A_1)$  calibrator  
 X Calibrator concentration = IU/mL ASO

#### QUALITY CONTROL

Control sera are recommended to monitor the performance of manual and automated assay procedures.

Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

#### NORMAL RANGE

Up to 200 IU/mL (adults) and 100 IU/mL (children < 5 years old), Each laboratory should establish its own reference range.


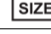

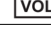

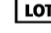





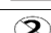

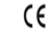
#### PERFORMANCE CHARACTERISTICS

- Linearity limit:** Up to 800 IU/mL, under the described assay conditions. Samples with higher concentrations, should be diluted 1/3 in NaCl 9 g/L and retested again. The linearity limit depends on the sample-reagent ratio, as well the analyzer used. It will be higher by decreasing the sample volume, although the sensitivity of the test will be proportionally decreased.
- Detection limit:** Values less than 20 IU/mL give non-reproducible results
- Prozone effect:** No prozone effect was detected up to 1000 IU/mL
- Sensitivity:**  $\Delta$  0.73 mA. IU/ML
- Precision:** The reagent has been tested for 20 days, using three different ASO concentrations

EP5	CV (%)		
	$\pm$ 100 IU/mL	$\pm$ 200 IU/mL	$\pm$ 400 IU/mL
Total	6.4%	5.7%	5.1%
Within Run	2.4%	1.7%	1.4%
Between Run	3.6%	4.2%	4.9%
Between Day	4.7%	3.5%	0.7%

- Accuracy:** Results obtained using this reagent (y) were compared to those obtained using a commercial reagent (x) with similar characteristics. 60 samples of different concentrations of ASO were assayed. The correlation coefficient (r) was 0.99 and the regression equation  $y = 0,915x - 4,844$ . The results of the performance characteristics depend on the analyzer used.
- NOTES**  
Clinical diagnosis should not be made on findings of a single test result, but should integrate both clinical and laboratory data.
- INTERFERENCES**  
Bilirubin (20 mg/dL), hemoglobin (10 g/L), lipemia (10 g/L) and rheumatoid factors (600 IU/mL), do not interfere. Other substances may interfere.
- NOTES**  
Clinical diagnosis should not be made on findings of a single test result, but should integrate both clinical and laboratory data.

#### SYMBOL ON LABELS

Symbols	Signify	Symbols	Signify
	Catalogue Number		Pack Size
	Expiry Date		Volume
	Storage Condition		Lot Number
	Instruction for Use		In Vitro Diagnostics
	Manufacturing Date		Manufacturer
	Number of Tests		For Single Use Only
	EC Representative		European conformity

#### REFERENCES

- Young DS. Effects of drugs on clinical laboratory test, 4th ed. AACC Press,1995
- Haffejee I, Quarterly Journal of Medicine 1992, New series 84:305-641
- Alouf Jodeph E, Pharma Ther 1980; 11: 661-717.
- M Fasani et al. Eur J Lab Med 1994; vol2.n°1: 67.
- Todd EW.J Exp Med 1932; 55:267-280
- Klein GC. Applied Microbiology 1971; 21: 999-1001.
- Young DS. Effects of drugs on clinical laboratory test, 4th ed. AACC Press,1995
- Klein,GC.Applied Microbiology 1970;19:60-61